

Start	Exams	Anatomies	Settings	Tools	Positioning	Acquisition		
<p>A</p>	<p>1</p> <p>PAN</p>	<p>2</p> <p>ADULT CHILD</p>	<p>3</p> <p>ADULT CHILD</p> <p>ECO ECO</p> <p>REGULAR REGULAR</p> <p>ORTHO</p>	<p>3A TYPE</p> <p>AUTO</p> <p>SafeBeam</p>	<p>4</p>	<p>5 Device</p> <p>a Click OK</p> <p>Do you want to reset ?</p> <p>Dismiss Ok</p> <p>b automatic reset</p> <p>Device is moving</p> <p>Dismiss</p> <p>if necessary</p> <p>c</p>	<p>6 Patient</p> <p>if necessary</p> <p>POSITIONING INTENT</p>	<p>7 PAN - DENT - BTW - SIN - TMJ - CARPUS - CEPH LL, AP, PA</p> <p>a</p>
	<p>1</p> <p>DENT</p>	<p>2</p>	<p>3</p> <p>REGULAR</p>	<p>3A TYPE</p> <p>MANUAL</p> <p>← →</p> <p>← →</p>	<p>4</p>	<p>5 Device</p> <p>a Click OK</p> <p>Do you want to reset ?</p> <p>Dismiss Ok</p> <p>b automatic reset</p> <p>Device is moving</p> <p>Dismiss</p> <p>if necessary</p> <p>c</p>	<p>6 Patient</p> <p>if necessary</p> <p>POSITIONING INTENT</p>	<p>a</p>
	<p>1</p> <p>BTW</p>	<p>2</p>	<p>3</p> <p>REGULAR</p>	<p>3A TYPE</p> <p>MANUAL</p> <p>← →</p> <p>← →</p>	<p>4</p>	<p>5 Device</p> <p>a Click OK</p> <p>Do you want to reset ?</p> <p>Dismiss Ok</p> <p>b automatic reset</p> <p>Device is moving</p> <p>Dismiss</p> <p>if necessary</p> <p>c</p>	<p>6 Patient</p> <p>if necessary</p> <p>POSITIONING INTENT</p>	<p>a</p>
	<p>1</p> <p>SIN</p>	<p>2</p> <p>SIN L SIN CENT SIN R</p>	<p>3</p> <p>REGULAR</p>	<p>3B MODE</p> <p>Mode: __</p> <p>Exposure Time: __</p> <p>Estimated DAP (mGy x cm²): __</p>	<p>4</p>	<p>5 Device</p> <p>a Click OK</p> <p>Do you want to reset ?</p> <p>Dismiss Ok</p> <p>b automatic reset</p> <p>Device is moving</p> <p>Dismiss</p> <p>if necessary</p> <p>c</p>	<p>6 Patient</p> <p>if necessary</p> <p>POSITIONING INTENT</p>	<p>b</p>
	<p>1</p> <p>TMJ</p>	<p>2</p> <p>TMJ FRONT TMJ LAT</p>	<p>3</p> <p>REGULAR</p>	<p>3B MODE</p> <p>Mode: __</p> <p>Exposure Time: __</p> <p>Estimated DAP (mGy x cm²): __</p>	<p>4</p>	<p>5 Device</p> <p>a Click OK</p> <p>Do you want to reset ?</p> <p>Dismiss Ok</p> <p>b automatic reset</p> <p>Device is moving</p> <p>Dismiss</p> <p>if necessary</p> <p>c</p>	<p>6 Patient</p> <p>if necessary</p> <p>POSITIONING INTENT</p>	<p>b</p>
<p>D</p> <p>iPad</p>	<p>Exam-CEPH</p> <p>1</p> <p>CARPUS</p>	<p>2</p>	<p>3</p> <p>REGULAR</p>	<p>3A TYPE</p> <p>AUTO</p> <p>SafeBeam</p>	<p>4</p>	<p>5 Device</p> <p>a Click OK</p> <p>Do you want to reset ?</p> <p>Dismiss Ok</p> <p>b automatic reset</p> <p>Device is moving</p> <p>Dismiss</p> <p>if necessary</p> <p>c</p>	<p>6 Patient</p> <p>if necessary</p> <p>POSITIONING INTENT</p>	<p>c</p>
<p>E</p>	<p>1</p> <p>LATERAL</p>	<p>2</p> <p>FULL LONG FULL STANDARD</p> <p>REDUCED LONG REDUCED STANDARD</p>	<p>3</p> <p>REGULAR</p> <p>ECO</p>	<p>3A TYPE</p> <p>MANUAL</p> <p>← →</p> <p>← →</p>	<p>4</p>	<p>5 Device</p> <p>a Click OK</p> <p>Do you want to reset ?</p> <p>Dismiss Ok</p> <p>b automatic reset</p> <p>Device is moving</p> <p>Dismiss</p> <p>if necessary</p> <p>c</p>	<p>6 Patient</p> <p>if necessary</p> <p>POSITIONING INTENT</p>	<p>c</p>
<p>1</p> <p>AP-PA</p>	<p>2</p> <p>AP PA</p>	<p>3</p> <p>REGULAR</p>	<p>3B MODE</p> <p>Mode: __</p> <p>Exposure Time: __</p> <p>Estimated DAP (mGy x cm²): __</p>	<p>4</p>	<p>5 Device</p> <p>a Click OK</p> <p>Do you want to reset ?</p> <p>Dismiss Ok</p> <p>b automatic reset</p> <p>Device is moving</p> <p>Dismiss</p> <p>if necessary</p> <p>c</p>	<p>6 Patient</p> <p>if necessary</p> <p>POSITIONING INTENT</p>	<p>c</p>	