


NewTom X-VS IMMEDIATE.VISION

ALWAYS READY, FAST, PORTABLE



TOTAL SYNERGY

The X-VS means real-time diagnostics, direct USB plug-and-play connection, high definition and immediate results. The X-VS uses NNT, the all-in-one software ideal for diagnostics, communication and management of intraoral imaging; perfect for storing, managing and printing images in perfect synchronism with any other device already in the surgery.

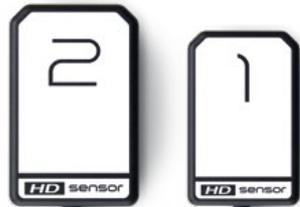


DUAL FORMAT

The X-VS is equipped with a sensor, available in two sizes - ergonomic and with smoothed edges - which adapt it to the anatomy of the patient's oral cavity, guaranteeing outstanding positioning comfort. Reduced dimensions and a maximum active area ensure advanced X-ray diagnosis.



FIVE-LAYER SENSOR

Five-layer technology, for high-contrast images and precision detail. Cesium Iodide (CsI) scintillator made up of column-like micro-structures that preserve image quality; it first intercepts the X-ray beam and converts it into visible light. The fibre optics layer (Fibre Optics Plate) collimates the radiation onto the sensor and protects it against direct X-ray penetration. The third layer (CMOS HD) is the acquisition device which converts the light into a digital image with 16,384 grey levels. The fourth layer pre-processes the image and converts it into a USB signal. The last layer has a protective function.

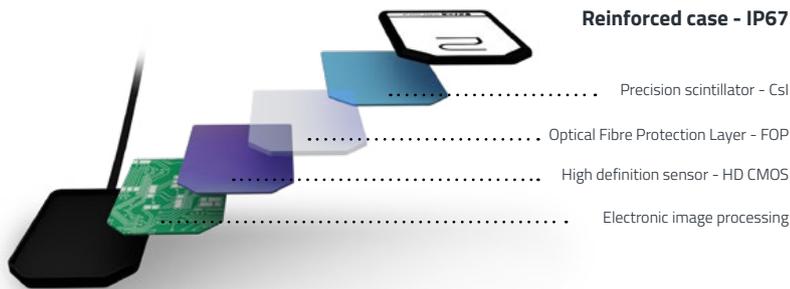

X-VS Sensor	Size 1 - Regular	Size 2 - Large
External dimensions (mm)	38.9 x 24.9	41.9 x 30.4
Thickness (mm)	5.3	5.7
Pixel matrix	1500 x 1000	1700 x 1300
Pixel size (µm)	20	20
Maximum resolution (lp/mm)	25	25
Grey levels depth	14-bit acquisition - 16384 maximum grey levels	
Scintillator technology	CsI (Cesium Iodide) with micro-columnar structure	
Direct exposure protection	FOP (Fibre Optics Plate)	
Protection rating	IP 67 (Guaranteed against liquid or dust infiltration)	
Compatibility with X-ray generators	Any AC or DC technology X-ray generator with kV values in the 60-70 kV range and precision control of exposure times	
Connectivity	Direct USB to PC	
Acquisition software (for PC)	iCapture with TWAIN interface	
Image management software (for PC)	NNT (for PC) with DICOM 3.0 interface with free viewer and APP for iPad	
Minimum system requisites		
Supported operating systems	Microsoft® Windows® 10 Professional 64 bit	
Display settings	1280 x 1024; 1344 x 768 or greater, 16 million colours	
Port	USB 2.0 or subsequent	
Power supply	5 VDC, 500 mA (via USB)	



BU Medical Equipment

**Sede legale ed amministrativa
 Headquarters**

CEFLA s.c.
 Via Selice Provinciale 23/a ▪ 40026 Imola ▪ Italy
 t. +39 045 8202727 ▪ 045 583500
 info@newtom.it

**Stabilimento
 Plant**

Via Bicocca, 14/c
 40026 Imola ▪ Bo (Italy)
 tel. +39 0542 653441
 fax +39 0542 653601

newtom.it

